

Check-Spot - What's that?



That's the Check-Spot when it is delivered to the blood bank – a self adhesive, temperature sensitive label, not yet activated. A slight pink coloration of the indicator before activation may occur, but is entirely insignificant.

Careful!
Do not touch the indicator!




That's the Check-Spot when activated and working, indicated by the green dot.

Please note:

Ensure that further transport after having been conveyed by a pneumatic tube, be effected under chilled conditions!

As long as the O in the centre remains white (or light pink) the core temperature of the blood bag (RBCs) has never reached 10° C, was therefore stored in accordance with the regulations and RBCs may be transfused. In case of non-usage, please return the blood bag to the blood bank (blood deposit) for further storage.

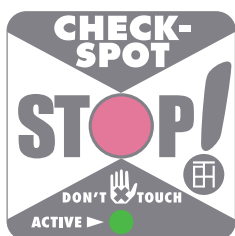

That's the Check-Spot when core temperature of the blood bag (RBCs) has exceeded 10° C. The red color of the O indicates that regardless of whether blood bag was chilled afterwards, an immediate transfusion is still permissible but no further storage. In case of non-usage, please return the bag to the blood bank (blood deposit) for waste disposal.



Harald H. Temmel KG
A-8200 Gleisdorf
Kernstockgasse 4
Tel. 03112-2917
Fax 03112-4371

Email: office@temmel-austria.com